melcare™

# OPTIMEL<sup>®</sup> MANUKA<sup>+</sup> DRY EYE DROPS

Instructions For Use  
Mode d'emploi  
Instrucciones de uso  
Gebrauchsanleitung

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IFU0344  
V02.08.2022

## EN Instructions For Use

**OPTIMEL Manuka+ Dry Eye drops**  
Can improve eye comfort and reduce redness by improving tear quality. For use as an adjunct therapy in addition to good eyelid hygiene and lubricants.

**INDICATIONS**  
May reduce eye discomfort due to chronic dry eye.

**DIRECTIONS**  
Wash hands.  
After first removing cap, discard tamper ring on bottle. Avoid touching any surface with nozzle.

Instil 1 drop to the inner corner of the affected eye up to 3 times daily or as directed by your eye care professional.

Discard 2 months after opening.

**PRECAUTIONS**  
Single patient use only. Serious conditions should be managed under the supervision of an eye care professional.

DO NOT use if sensitive to honey or any of the ingredients.

DO NOT use if packaging is damaged or tamper ring is not attached to cap base – remove cap and discard ring.

DO NOT instil whilst wearing

contact lenses. Wait at least 10 minutes before inserting lenses.

Avoid use if your eyes are very inflamed unless otherwise advised by your eyecare professional.

Transient stinging and redness are expected.

Preservative free lubricant may be used beforehand to reduce sting, especially when eyes are very dry.

For any reaction other than transient stinging and redness, cease use and contact your eye care professional.

Seek advice from an eye care professional for children under 16.

Discard 2 months after opening.

Replace cap after use.

**INGREDIENTS**  
Honey (16.5%), sodium chloride, glycerol, sodium benzoate (preservative).

**HOW SUPPLIED**  
Supplied in a dropper bottle with ophthalmic nozzle.

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