melcare<sup>+</sup>

# OPTIMEL<sup>®</sup> MANUKA<sup>+</sup> FORTE EYE GEL

## Instructions For Use

## Mode d'emploi

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Melcare Biomedical Pty Ltd  
451 West Mt Cotton Rd  
MOUNT COTTON QLD 4165  
AUSTRALIA

Melcare (Europe) Ltd  
Carpenter Court  
1 Maple Rd, Bramhall  
STOCKPORT SK7 2DH  
ENGLAND

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## EN Instructions For Use

### OPTIMEL Manuka+ Forte Eye Gel

Can improve eye comfort and reduce redness by improving tear quality. For use as an adjunct therapy in addition to good eyelid hygiene and lubricants.

#### INDICATIONS

May reduce eye discomfort due to chronic dry eye. Ideal for use prior to sleep.

#### DIRECTIONS

Wash hands. Remove cap, do not allow nozzle to touch any surface.

Instil 1 drop to the inner corner of the affected eye OR dispense onto tip of cotton bud and gently apply along lower lid margin in evening and morning or as directed by your eye care professional.

Discard 2 months after opening.

#### PRECAUTIONS

Single patient use only. Serious conditions should be managed under the supervision of an eye care professional.

DO NOT use if sensitive to honey or any of the ingredients.

DO NOT use if packaging is damaged or carton seal is not in place before first use.

DO NOT instil whilst wearing contact lenses. Wait at least 10 minutes before inserting lenses.

Transient stinging and redness are expected. For any reaction other than transient stinging and redness, cease use and contact your eye care professional.

Seek advice from an eye care professional for children under 16. Discard 2 months after opening.

Replace cap after use.

#### INGREDIENTS

Honey (98%), gum (Acacia senegal), glycerol.

PRESERVATIVE FREE.

#### HOW SUPPLIED

Supplied in a tube with ophthalmic nozzle.

Tube is oversized to allow contents to expand during sterilisation.

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